


TEETH WHITENING GEL HOME KIT



teewite

Brighten up your smile

DIRECTIONS FOR USE

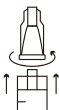

Twist to remove gel lid
before applying gel



Make sure you spread
the gel evenly in the tray



Apply the gel to the
tray along the line



Whiten for the requested
duration. Rinse and Repeat as
per your dentist's indications

35% Hydrogen Peroxide

IFU - SCAN ME

CAUTION



- 1- Not suitable for ceramic or false teeth.
- 2- Not suitable for teeth discoloration caused by lesion or medicine.
- 3- Not suitable for children under 18 years old and pregnant women.
- 4- To be sold to dental practitioners.

TEETH WHITENING GEL HOME KIT

Medibrex Hong Kong Headquarters

Librex Hong Kong
RM 907 Silvercord, Tower 2, 30 Canton
RD, Tsim Sha Tsui Kowloon, Hong Kong

Medibrex Middle East Headquarters

Medibrex Offshore
Arz Hospital str., Librex Bldg, Zalka

Medibrex British Virgin Islands Headquarters

Librex, inc.
(BUI) No: BC 1057776- Road Town,
Tortola- British Virgin Islands



info@medibrex.com

Ingredients: Propylene Glycol, Glycerin, Water, Carbomer, Cellulose Gum, Sodium Bicarbonate, PVP, Menthol, Sodium Hydroxide, Hydrogen Peroxide.

Package contents: 4 x Syringes of Teewite whitening Gel, 35% Hydrogen Peroxide (3ml each)

MADE IN PRC

teewite

B r i g h t e n u p y o u r s m i l e

OFFICE KIT

35% Hydrogen Peroxide



TEETH WHITENING GEL - OFFICE KIT

INGREDIENTS:

TEEWITE 35% HP Teeth Whitening Gel

Active ingredient: Hydrogen Peroxide 35%

Inactive ingredients: Propylene Glycol, Glycerol, Water, Carbomer, Carboxymethylcellulose Sodium.

IFU - SCAN ME



TEEWITE 58% Light Cured Gum Protector

Active ingredient: Diurethane Dimethacrylate 58%

inactive ingredients: Silicon Dioxide, Methacryloxyethyl Trimellitic Anhydride, Titanium Dioxide, Light Initiators, Sodium Saccharin, Pigments.

WARNINGS:

For professional use only - Harmful if swallowed - Causes skin irritation Causes serious eye irritation - Keep out of reach of children - Wash hands thoroughly after handling.

Warnings:

For professional use only - Harmful if swallowed -Keep out of reach of children.



Medibrex Hong Kong Headquarters

Librex Hong Kong
RM 907 Silvercord, Tower 2, 30 Canton
RD, Tsim Sha Tsui Kowloon, Hong Kong

Medibrex Middle East Headquarters

Medibrex Offshore
Arz Hospital str., Librex Bldg, Zalka

Medibrex British Virgin Islands Headquarters

Librex, inc.
(BUI) No: BC -1057776 Road Town,
Tortola- British Virgin Islands



info@medibrex.com

Contents: Dual-barrel, 2.5ml, clinical whitening gel x 2;
Light cured gum protector gel 1.5ml x 2; replace needles x 4

MADE IN PRC